GOCOZY EYE HEAT PATCHES

Indications

Use for warm compresses in cases of dry eyes, blepharitis, Meibomian Gland Dysfunction (MGD), chalazion, and sty.

Use for relaxation, relieving eye strain and fatigue caused by prolonged use of phone and computer screens, and for people with computer vision syndrome.

Important Safety and Use Information

PLEASE READ CAREFULLY THE INSTRUCTIONS FOR USE AND ALL WARNINGS BEFORE USING THIS PRODUCT. TO REDUCE THE RISK OF BURNS AND PERSONAL INJURY, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE USE INSTRUCTIONS, WARNINGS, AND PRECAUTIONS.

Instructions for Use

1. Tear the pouch. The eye mask will warm up automatically upon opening of the pouch.
2. Place the eye mask over both eyes with eyelids gently closed.
3. Relax and feel comfortable with your eyes gently closed. For maximum effectiveness, wear the eye mask for at least 10 minutes.
4. Use once or twice a day for general application. For conditions like chalazion and sty, apply four times a day.

If you experience any discomfort, burning, swelling, rash, or other changes on your skin where the eye mask was worn:

STOP AND CALL YOUR DOCTOR IMMEDIATELY.

Precautions and Warnings

- Contact lenses/Eye surgery: remove contact lenses or other devices prior to use. If you have had recent cataract or other eye surgery, ask your eye care specialist before use.
- If you experience irritation, discomfort, or burning, remove the Gocozy eye heat patches immediately.
- For all consumers:
 - Gocozy is to be used with eyelids closed.
 - Do not use on unhealthy, numb, damaged, or broken skin.
 - Do not use after eyelid surgery or eye surgery without approval from your eye surgeon.
 - Do not microwave or attempt to reheat. Gocozy eye heat patches are for one-time use only. Discard appropriately after use.
 - Do not use if eyelids do not close normally.
- Additional important considerations: if you are concerned, consult your doctor as they are most familiar with your health.

Storage Conditions


Store at room temperature.
Keep out of reach of children and pets.

The owner of the medical device registration number:

Vimedlink Medical Co.LTD.,
Address: lot 10, Group 9, Viet Hung street, Viet Hung ward, Long Bien district, Hanoi, Vietnam.

Marketing and Distribution

GoMed Medical Co.LTD.,
Address: 352 Giai Phong street, Phuong Liet ward, Thanh Xuan district, Hanoi, Vietnam.
<https://gomed.vn>

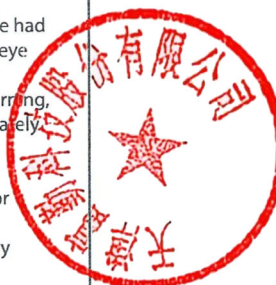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