CLIENTE / Customer :	FARMIGEA
BOZZA DEL / Proof date : N° 4	del 18 - 11 - 2020
PRODOTTO / Product : Hye MSC Lett EN-Viet	CODICE / Code : 912917/00
Cod. LAETUS IXXIIXXIIIX (2864)	G/MQ / g/smq : 50
FTO POSA STESA / Size flat : 135 x 220mm	FTO POSA PIEGATA / Size folded : 135x28mm
STAMPA SU LATO / Printing : BIANCA / Top side :	☒ VOLTA / Reverse side : ☒
COLORI / Colors : NERO - FUSTELLA	
NOTE / Notes :	C. Laetus: I 0,35x6mm / X 1x6mm / I-I 0,65mm

912917/00

Sodium Hyaluronate 0.4%

Lubricating Eye Drops
10 ml preservative free bottle

Composition: Sodium Hyaluronate (0.4% w/v), sodium chloride, sodium citrate, citric acid monohydrate, water for injections.

What is it?

Hye is a sterile, preservative free eye drops solution of Sodium Hyaluronate.

Phosphate and preservative free.

How is it presented?

Hye is presented as an eye drops solution in a preservative free bottle of 10 ml.




When should it be used?

Hye is an ophthalmic solution of high-molecular weight Sodium Hyaluronate, obtained by biotechnological synthesis. Sodium Hyaluronate is a substance that originates mucomimetic solutions, with optimal lubricant properties due to the viscosity of the solutions that result in a moisturising effect and stabilization of the tear film.

The lubricant, moisturising and protective qualities of sodium hyaluronate relieve irritation, dryness, burning and foreign body sensation due to environmental elements (wind, sun, smoke, dry air), visual stress (excessive light and prolonged use of computers) or mechanical factors (contact lenses use, after ocular surgery), providing a long lasting relief. The product is preservative free and suitable for use with contact lenses.

How much to use?

1 drop in the eye one or more times daily, according to necessity.

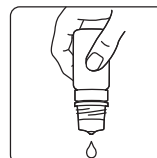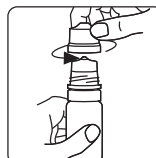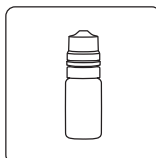Precautions and warnings

- The product is for external ophthalmic use only.
- Do not use if the container is damaged.
- Do not use if you are sensitive to any of the components.
- Do not use after the expiry date shown on the container.
- Do not touch the eye with the tip of the bottle.
- After administration, in rare cases, a slight temporary blurring of vision may be observed due to the viscosity of the solution; it is therefore advisable to wait until the blurring has disappeared before any activity that requires clear vision.
- Leave at least 10-15 minutes before administration of any other ophthalmic drugs.
- Once opened, each bottle can be used for up to 90 days.
- Carefully close the bottle after each use, ensuring the tip is free from any residual drops; if present, shake any off with a sharp wrist movement.
- Keep out of the sight and reach of children.
- Store below 25°C.
- Do not swallow.
- To avoid the risk of bottle collapsing, do not deliver more than 30 consecutive drops.

Manufacturer: Farmigea S.p.A. - Via G.B. Oliva, 6/8 - 56121, Pisa, Italy

Import and distribution: GoMed Medical Co., LTD

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Website: <https://gomed.vn>

Date of last revision of the text: April 2020

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Nhà sản xuất: Farmigea S.p.A. - Via G.B. Oliva, 6/8 - 56121 Pisa, Italy