CLIENTE / Customer :	FARMIGEA
BOZZA DEL / Proof date : N° 3	del 21 - 10 - 2020
PRODOTTO / Product : Hye Mono Lett EN-Viet	CODICE / Code : 912918/00
Cod. LAETUS IXXIIXXI (2865)	G/MQ / g/smq : 50
FTO POSA STESA / Size flat : 135 x 220mm	FTO POSA PIEGATA / Size folden : 135x28mm
STAMPA SU LATO / Printing : BIANCA / Top side :	☒ VOLTA / Reverse side : ☒
COLORI / Colors : NERO - FUSTELLA	
NOTE / Notes :	C. Laetus: I 0,35x6mm / X 1x6mm / I-H 0,65mm

912918/00

Hye mono

FARMIGEA

Sodium Hyaluronate 0.4%

Moisturizing, lubricant ophthalmic solution

20 preservative free reclosable mini-containers of 0.5 ml

Composition: Sodium Hyaluronate (0.4% w/v), sodium chloride, sodium citrate, citric acid monohydrate, water for injections.

What is it?

Hye mono is a sterile ophthalmic solution containing Sodium Hyaluronate.
Phosphate and preservative free.

How is it presented?

Hye mono is presented as an eye drops solution in 20 reclosable vials of 0.5 ml.

When should it be used?


Hye mono is an ophthalmic solution of high-molecular weight Sodium Hyaluronate, obtained by biotechnological synthesis. Sodium Hyaluronate is a substance that originates mucomimetic solutions, with optimal lubricant properties due to the viscosity of the solutions that result in a moisturising effect and stabilization of the tear film.

The lubricant, moisturising and protective qualities of sodium hyaluronate relieve irritation, dryness, burning and foreign body sensation due to environmental elements (wind, sun, smoke, dry air), visual stress (excessive light and prolonged use of computers) or mechanical factors (contact lenses use, after ocular surgery), providing a long lasting relief.

How and how much should it be used?

- Open the aluminum bag and separate one mini-container from the strip of mini-containers;
- Open it by turning and pulling the cap;
- Put one drop in the eye by squeezing gently the body of the mini-container, one or more times daily, according to necessity.

Hye mono is filled in re-closable vials and the remaining product can be used up to 12 hours. **Hye mono** is compatible with contact lenses and it is not necessary to remove them before the ophthalmic solution is applied.

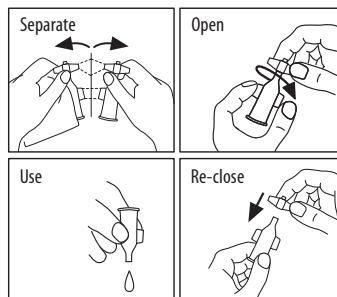Precautions and warnings

- The product is for external ophthalmic use only.
- Do not use if the mini-container is damaged.
- Do not use in case of known incompatibility to any of the ingredients of the preparation.
- Do not use after the expiry date shown on the pack.
- Do not touch the eye with the tip of the mini-container.
- After administration, in rare cases, a slight temporary blurring of vision may be observed due to the viscosity of the solution; it is therefore advisable to wait until the blurring has disappeared before any activity that requires clear vision.
- Leave at least 10-15 minutes before administration of any other ophthalmic drugs.
- Once opened, each vial can be used for up to 12 hours.
- Keep out of the sight and reach of children.
- Store below 25°C.
- Do not swallow.

Manufacturer: Farmigea S.p.A. - Via G.B. Oliva, 6/8 - 56121 Pisa, Italy

Import and distribution: GoMed Medical Co., LTD

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Website: <https://gomed.vn>

Date of last revision of the text: April 2020

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Nhà sản xuất: Farmigea S.p.A. - Via G.B. Oliva, 6/8 - 56121 Pisa, Italy