OCUVANE® WARM

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THIẾT BỊ Y TẾ

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Steam Eye Mask

MEDICAL DEVICE

OCUVANE[®] WARM warms up when opened and produces optimal 45°C steam heat in approximately 30 minutes.

New technology with a double-layer steam mask containing menthol helps significantly increase the volume of tears secreted, tear film breakup time as well as improve meibum condition in healthy people and dry eye patients.

Instant warmth quickly warms your eyes, helps you relax and brightens your eyes.

Innovative non-woven fabric allows for a closer fit around the eye area for a warmer feel.

The ear strap is anti-slip and caters for use in anywhere.

APPLICATIONS

- Use for warm compresses in cases of dry eyes, blepharitis, Meibomian gland dysfunction (MGD), chalazion, styes.
- Relax, reduce eye strain caused by using a lot of phone and computer screens.

INGREDIENTS

Iron Powder, Activated Carbon, Vermiculite, Water, Salt, Semen Cassiae, Wild Chrysanthemum, Mulberry Leaves, Menthol.

HEBEI HOUDE HANFANG MEDICAL DEVICES GROUP CO., LTD

Beisun Village, Zhengding County, Shijiazhuang City, Hebei Province, P.R.C

Distributor

SOLMED PHARMA KFT.

19 Ady Endre Street, 1024 Budapest, Hungary

INSTRUCTION

STEP 1: Remove eye mask from the pouch. Use immediately upon opening as the mask begins to warm up instantly.

STEP 2: Gently tear along perforation, then place elastic bands over ears and white area of the mask over eyes.

STEP 3: Relax about 30 minutes until the eye mask no longer warm.

STEP 4: Remove mask and dispose it in the trash. Don't reuse.

WARNING

Use as directed. **FOR EXTERNAL USE ONLY.**

Harmful if swallowed, contains iron. Do not put in a microwave. Do not cut the eye mask. Do not use damaged mask. Eye mask will not warm up if the packing bag is scratched or torn. If you have eye problems, eye diseases, injury around the eyes, allergic reactions, pregnant - please contact your doctor before use. In case of any discomfort or allergic reaction, please stop using immediately and rinse out the areas with cool water.

Please remove contact lenses before use.

Store at room temperature. Keep out of the reach and sight of small children.

BASIC UDI-DI 599988714OCUVANEU3

OCCUVANE[®] WARM

Steam Eye Mask

A5 (14.85 x 21.0 cm)

447C