ORTOPAD®

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Reference Member State.

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Tel: 32 (0) 2 732 5954
Fax: 32 (0) 2 732 6003
Email: mail@obelis.net

APPLICATION INSTRUCTIONS

To ensure proper patching results, the Ortopad patch must be applied as follows:

- 1 • First, clean and dry the skin around the eye to be patched.
- 2 • Carefully open the individual package and separate the paper liner from the eye patch.
- 3 • Gently close both eyes. Keep facial muscles relaxed. Do not close eyes tightly.
- 4 • Place the eye patch STRAIGHT over the eye to be covered, with the narrow end towards the nose. For best patching results, it is very important to have the inlay of the patch straight in front of the eye.
- 5 • Gently press the patch in place. Lightly rub over the patch a few times. Pay special attention to the area closest to the nose. The Ortopad patch has a special temperature sensitive adhesive that works best at body temperature.
- 6 • Wear and replace the patch according to your health professional's direction.

✧ **Removing the patch:** carefully peel off the edges of the patch and gently pull the patch away from the skin.

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PIETRASANTA PHARMA S.p.A.
Obelis UK Ltd